


assistina

Weekly checklist


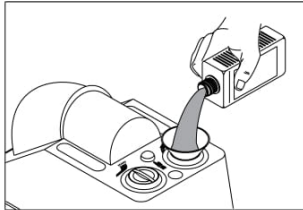


**Check liquid level!
Refill if necessary!**



Service oil:
500 ml – 02675930

Cleaning solution:
1000 ml – 02680230


Set 02680500:
1000 ml Cleaning solution and 500 ml Service Oil

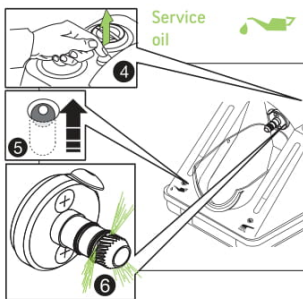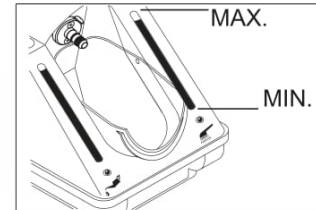

Check function!

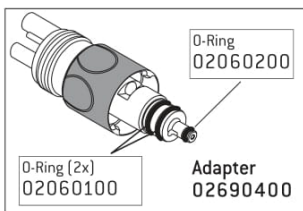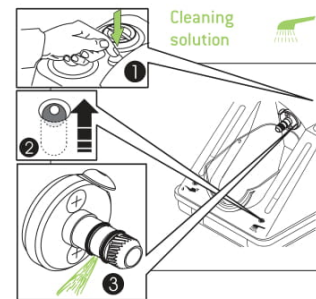


**Check o-rings!
Look for right position
and / or
damage!**

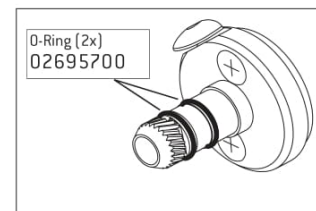



General hints



When maintaining after every patient – Press program button once!
When maintaining twice a day – Press program button twice!

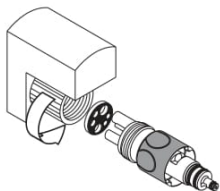

Check connection is tight!

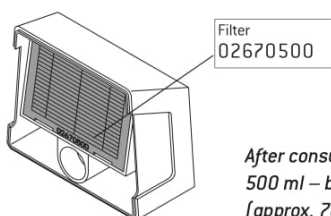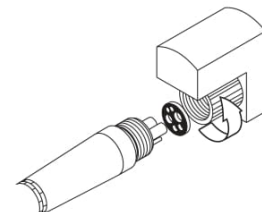

After consumption of one
500 ml – bottle of Service oil
(approx. 7000 cycles)

Changing of filters!



Once a year

8/21/2020

Ignaz-Glaser-Straße 53, P.O. Box 1
5111 Bürmoos, Austria



ISO 15403
93/42/EEC – Annex II

Assistina weekly checksheet - hayeshandpiece.com.au

f +43 (0) 6274 6236-710
e techn.support@wh.com
wh.com

Form-Nr. 5